



Nasir NEB is a device for the administration of topical solutions through intranasal nebulization, such as anti-inflammatory, analgesic, anaesthetic, antihistamine and anticonvulsant drugs, in hospital or home settings.

INTENDED USE

It is also indicated for the treatment of nasal cavities, paranasal sinuses and upper airways for the administration of nasal washes.

USE ENVIRONMENT

The device is intended to be used in any environment.

USERS

Indicated for infants, children and adults.

CONTRAINDICATIONS

Do not use if you have severe ear or upper respiratory infections, severe nasal congestion, cold symptoms and a perforated eardrum.

USE RESTRICTIONS

The device is intended for single patient use.

After use, rinse the nozzle inside and out with drinking water and store the device in a dry place, away from light and heat sources.

For subsequent reuse, first disinfect by immersing the nozzle in AMUCHINA® 2% solution (20 ml of product in one litre of water).

Let the disinfectant work for 15 minutes, then rinse the nozzle thoroughly inside and out with drinking water.

By following these instructions, the nozzle can be used for a maximum of 100 deliveries, within 1 month of package opening.

Use the product only for its intended use.

After use, do not disperse into the environment and dispose of it as common urban waste.

WARNINGS

- Do not use Nasir NEB after the expiry date stated on the package.
- Do not use if package or device is open or damaged.
- Before use, consult a qualified medical professional.
- Do not use in case of epistaxis (nose bleed) in progress or just experienced.
- Do not use in case of specific hypersensitivity to any component.
- The assistance of an adult during use is recommended for treatment delivered to disabled and elderly patients, and children.
- Contains small and dangerous parts (nozzle and needle): keep out of the reach of children and animals.
- The product is for external use only.
- Do not force the device into the nasal cavity.
- Do not sterilize in autoclave.
- After use, do not disperse into the environment.
- Store away from direct sunlight and heat sources.
- Do not freeze.

HOW TO USE

- Attach the needle to the syringe and remove the needle cap
- Insert the syringe with needle into the vial or bottle of the solution to be nebulized and withdraw the liquid by pulling back the plunger of the syringe
- Hold the device upright with the syringe pointing up and remove the needle
- Keeping your chest upright, place the nebulizer nozzle on your nose and gently press on the nostril
- Press the plunger firmly to spray the solution
- If necessary, repeat with the other nostril

PACKAGE CONTENTS

1 NASIR NEB NEBULIZER NOZZLE

1 10 ML LUER LOCK SYRINGE

1 HYPODERMIC NEEDLE

INSTRUCTIONS

The patient and/or user must report any serious accident occurring during the use of the device; the report must be forwarded to the Ministry of Health according to the methods defined on the Ministry website and, for information, to E.P. Medica.

Rev. 01 of 30/06/2023

	Lot number
	Reference product code
	Do not use if package is broken or damaged
	CE marking according to REG 745-2017 requirements
	Generic hazard
	Read the instructions and warnings before use
	Manufacturer (producer)
	Date and country of manufacture
	Medical device

produced by E.P. Medica Srl
Via dell'artigianato 22
48034 Fusignano (RA)
+39 0545 1893255 | info@epmedica.it
www.epmedica.it