

ep medica



NASIR RAPID is a rapid and effective nasal irrigation device, suitable for daily hygiene by rinsing the nasal mucosa.

It prevents blockages due to conditions such as colds, coughs, sinusitis, post-operative conditions, itch, asthma, bronchitis, nasal polyps, allergens, or environmental factors (dust, smoke, or fumes).

Thanks to its L-shaped nozzle designed to simplify irrigation and limit syringe pressure, NASIR RAPID allows for safe nasal irrigation with a continuous flow of solution.

The L-shaped nozzle can be connected to any syringe with a Luer Lock connector.

POSODOGY

NASIR RAPID is indicated for children over 3 years of age and adults.

1-2 irrigations per day are recommended.

In case of very severe congestion, it is advisable to consult your doctor before proceeding with treatment.

USE INSTRUCTIONS

- Remove the solution bag from the plastic bag.



- Warm the solution bag to 35-37°C by immersing it in hot water.
- Remove the protective cap and screw on the syringe.
- Draw the solution from the bag and unscrew the syringe.
- Screw on the "L" nozzle.
- Tilt your head forward, lower than your shoulders.
- Position the nozzle to your nose and gently press on your nostril.
- Inject the solution until it comes out of the opposite nostril.
- Breathe through your mouth during this procedure.
- Repeat for the other nostril.
- It is advisable to blow your nose after rinsing.

If small amounts of the solution flow into the throat causing coughing, discontinue use and spit out or swallow.

The solution is non-toxic in small doses.

If the child experiences symptoms such as earache or a feeling of stuffiness, discontinue use.

Do not use if nosebleeds are ongoing or have just ended.

Do not use if you have a specific hypersensitivity to any component of the rinsing solution.

Do not use after the solution's expiration date.

Do not use if the device is damaged.

The product is for external use only.

The solution can be used within 48 hours of opening if stored properly in a dry place, away from light and heat sources.

LIMITATIONS

The sterility and expiration dates printed on the solution refer to the intact and properly stored product.

The device is intended for single-patient use and should be replaced periodically.

Nasir RAPID is produce by EP medica srl, via dell'artigianato 43/B – 48034 Fusignano (RA)

www.epmedica.it

Rev. 00 del 30/10/2025