




NASIR BABY IS RECOMMENDED
by midwives, pediatricians
and otolaryngologists.

soft nozzle for a
low pressure,
continuous flow
nasal wash

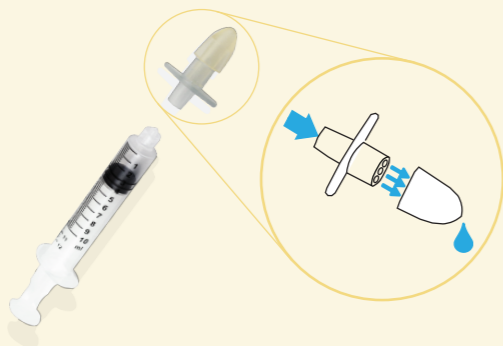

MADE IN ITALY
All the EP Medica products are made,
or undergo the last substantial processing in Italy.

Nasir[®]
BABY

Assemblato da E.P. Medica Srl
Via dell'artigianato 30
48034 Fusignano (RA)
C.F. e P.I. 02506400395
+39 0545 1893255 | info@epmedica.it
www.epmedica.it

Nasir[®]
BABY



Hygiene & Well-being
NASAL IRRIGATION
For daily nasal cleansing
0-36 MONTHS

Nasal wash designed for newborns, infants and toddlers up to 3 years old.

It favors the daily cleansing of the nose and the physiological breathing by removing infectious agents of the respiratory tract, which resulted from mucus stagnation.

It can prevent and cure **cold, sinusitis, otitis, bronchitis, rhinitis, allergy and oral respiration symptoms.**

This product is safe and without side effects, and it reduces the use of antihistamines and cortisone.

WHEN SHOULD YOU USE IT?

- before feeding
- before bedtime
- before using a nasal spray
- in case of allergies and/or asthma

OUR PROPOSAL ON THE MARKET

BABY pack of 3 patented anatomical nozzles
KIT 1 bag of saline solution (250ml)
1 syringe (10ml)
1 patented anatomical nozzle

RULES FOR AN EFFECTIVE WASH

TEMPERATURE

Our nose heats up the air that we breath. It is important to **cool down the solution between 35° and 38° C** to maintain the efficiency of the immune response.

POSITION

Lay the baby down with the head turned sideways and slowly insert the solution in the opposite nostril to the side on which he is lying. It is recommended to keep his mouth open.


PRESSURE

Dose the flow with the right pressure of the syringe piston. Too much pressure could cause a mucus' shift in the auditory tubes and cause otitis.

HOW TO MAKE A SUCCESSFUL WASH

Fill the syringe with the sterile solution. Lay the baby down on his side with the head to the side. Insert the anatomical nozzle on the syringe and place it in the upper nostril, then tilt the syringe so that it points toward the ear on the same side, to allow better mucus release. Insert the solution with a continuous, gentle and gradual movement.

The solution will come out the other nostril. Dry the nose. Lay the baby down on the other side and repeat the procedure in the second nostril. If the mucus is abundant, repeat the operation in both nostrils.

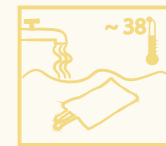

1 COOL DOWN

cool down the solution **between 35° and 38° C** before using it



2 DRAW

the solution with a syringe on which the anatomical nozzle has previously been applied



3 INSERT

the solution on the upper nostril while the baby is lying on his side with his head turned to one side. Repeat in the other nostril.

